

## Special Notice Regarding Virkon\*

The contents of this product have changed. Previously, a Virkon Rapid Pack was included for sanitisation of manikin faces during class. Due to problems in securing the supply of Virkon, this product will no longer be included with Laerdal manikins.

Instead, Laerdal Manikin Wipes are included in the following quantities:

Little Anne:	6 wipes	Resusci Baby	6 wipes	Resusci Anne:	50 wipes
Little Junior:	6 wipes	Resusci Junior:	18 wipes		

Virkon Rapid Packs (Catalogue number 154230xx \*\*10 x 5g tablets and bottle) and Tablets (Catalogue number 154231xx \*\*10 x 5g tablets) are available to purchase from Laerdal *only while stocks last*. These products will be discontinued when stocks run out.

\*This notice does not apply to manikins delivered in the US, Canada and Japan (language versions #####01 and #####05)

\*\*xx = Language version

## Sanitisation of Laerdal Manikin Faces

We recommend you provide an individual manikin face to each student. This eliminates the need for sanitisation between students.

All students can also use one manikin face permanently installed on the manikin and sanitise the manikin face between each student's use.

### Using the same manikin face for all students

If several students are using the same manikin face, thorough sanitation between students is required.

### To use Manikin Wipes

Tear foil packet open. Take out and unfold wipe. Rub manikin's mouth and nose vigorously. Wrap wipe snugly over mouth and nose. Let stay in place for 30 sec. Dry manikin's face with a clean paper towel or similar. Ventilation practice may now continue.

### Manikin Face Shields (optional)

Proper sanitisation of manikin's face provides proper hygienic conditions during training. However, for aesthetic reasons students may feel more comfortable having a clean barrier between own lips and surfaces touched by another person's lips. The Manikin Face Shields are designed to allow inexpensive barrier protection and to train the use of Laerdal Face Shields made for use in real cases. To use see instructions on package.

### Sodium Hypochlorite Solution

*Note: Wear gloves during sanitisation process (Use vinyl gloves. Latex gloves may stain manikin face).*

### After class:

1. Remove manikin face and detach the coupling from inside the face.
2. Clean and sanitise manikin faces and coupling
  - a. Wash with warm soapy water – scrub interior surfaces vigorously.
  - b. Rinse in clean water to remove soap residue.
  - c. Sanitising solution – Prepare a fresh sodium hypochlorite solution (at least 500ppm free available chlorine, i.e. >22ml of 2.5 – 5% household bleach per 500ml of water). This solution should only be used for sanitisation of a single class. Submerge the parts in the solution for at least 10 minutes. For alternative disinfectants, follow local policy.
  - d. Rinse with clean water to remove the solution residue.
3. Let parts dry.
4. Reassemble the face coupling and face skin. Push coupling through the mouth of the face skin until it is seated in the appropriate groove.
5. Remount face on manikin after reinstallation of airway.

### Disclaimer

Use of sanitation methods or materials other than those described above may result in less effective sanitation, and/or damage or discoloration of the Laerdal Manikin Faces. Laerdal therefore disclaims any responsibility for the effects of use of such non-recommended methods or materials.

If you have any questions about Manikin Face sanitisation or this product change please contact your local Laerdal customer service.